


# Tonometer Prism Tip Care (Cleaning / Disinfection)

The prism head of Goldmann-type tonometers should be cleaned immediately after use. The prism should either be soaked in a 1:10 sodium hypochlorite solution (household bleach), 3% hydrogen peroxide, or 70% isopropyl alcohol for 10 minutes. If a soaking solution is used, the prism should be rinsed and dried before reuse. If isopropyl alcohol is employed, it should be allowed to evaporate, or the prism head should be dried before reuse.

Although the use of alcohol is a recognized and well-established method of disinfection, its use may cause tension cracks in the tonometer tips. Haag-Streit strongly discourages the use of isopropyl alcohol 70%. Opti-Cide® is recommended for use as a disinfectant solution.

## Tonometer Tip Container 95387

with cap  
Holds 2 tonometer tips.  
Diameter: 2-1/8" (56 mm)

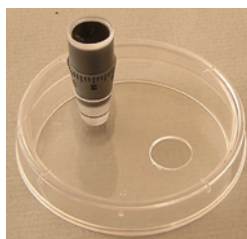

## Tonometer Tip Cleaning Stations


These tonometer tip cleaning stations have a screw cap and six ports to hold the tonometer tips. The top unscrews so that cleaning/disinfection solution may be added to the base. The three inside cleaning ports hold the used tonometer tips in the cleaning solution. Once cleaned and disinfected, the tips are moved to one of the three outside drying ports.

### Tonometer Tip Cleaning Station 95385

for GOLDMANN tonometer prisms  
with screw cap and drying ports  
Holds 6 tonometer tips Diameter: 3-1/8" (80 mm)


## Tonometer Tip Container 95383

with screw cap  
Holds 3 tonometer tips.  
2 ports for cleaning/disinfection,  
1 port for drying.  
Diameter: 1-3/4" (44 mm)

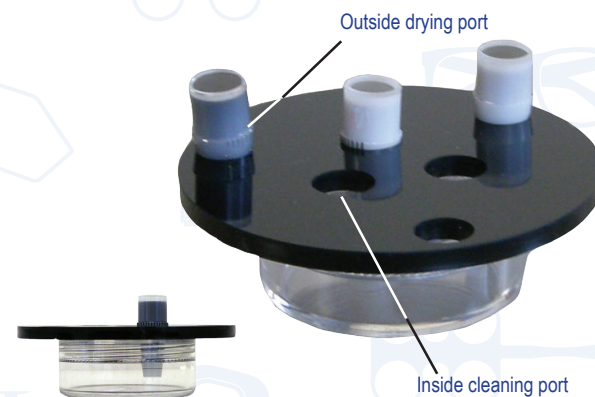

### Tonometer Tip Cleaning Station 95389

for CATS tonometer prisms  
with screw cap and drying ports  
Holds 6 tonometer tips Diameter: 3.5" (89 mm)

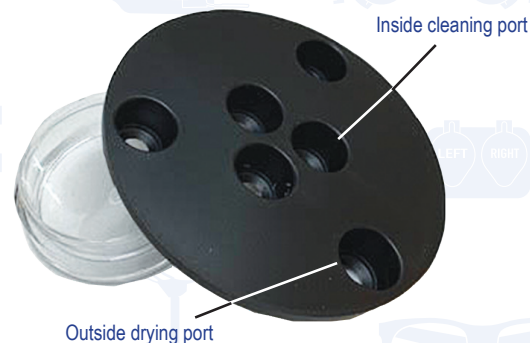

**OPTEGO / USA**  
3391 E Hemisphere Loop  
Tucson, AZ 85706  
Phone 866-678-6723  
Fax 520-209-1586  
email@optego.com  
www.optegousa.com

**OPTEGO / CANADA**  
3600B Laird Rd, Unit 3  
Mississauga ON L5L 6A7  
Phone 1 416-780-1289  
Toll Free 1 877-678-3464  
Fax 1 877-471-6001  
orders@optego.com  
www.optego.com

**OPTEGO / SINGAPORE**  
60 Paya Lebar Rd #11-33  
Paya Lebar Square  
Singapore 409051  
Phone +65 6742 5940  
asia@optego.com  
www.optego.com.sg